

# PAJUNK®



## *NerveGuard*

*Automatic Injection  
Pressure Limiter*



**CLINICAL  
BROCHURE**




## Causes and Approaches

# Avoiding Nerve Damage During Peripheral Nerve Blocks

Ultrasound-guided localisation of peripheral nerves and the associated real-time visualisation provide crucial benefits in Regional Anaesthesia.<sup>1</sup> However, it is clear that this does not reduce the incidence of permanent nerve damage.<sup>2</sup> Even in combination with nerve stimulation, intrafascicular injections cannot be ruled out.<sup>1,3,4</sup> The causes include incorrect needle position along with exceeding critical injection pressures.

Neurological complications during peripheral nerve blocks

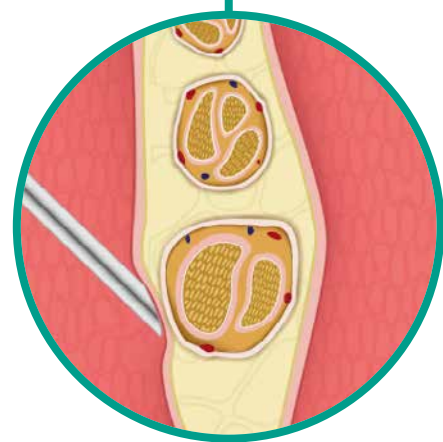



### Needle-Fascia Contact

**Effects:** Injections administered in the wrong tissue layers can cause anesthesia failures. A high opening pressure may indicate occlusion of the tip of the needle due to the fascia in front of it.<sup>8</sup>

→ One approach to avoid misinjections into wrong nerve tissue structures is to limit the opening pressure.

## Intrafascicular Injections

**Effects:** In the case of intrafascicular injection at high pressure lasting several hours, the microvascular blood supply of the nerve is severely restricted, which can lead to degeneration of nerve structures.<sup>6,11</sup>

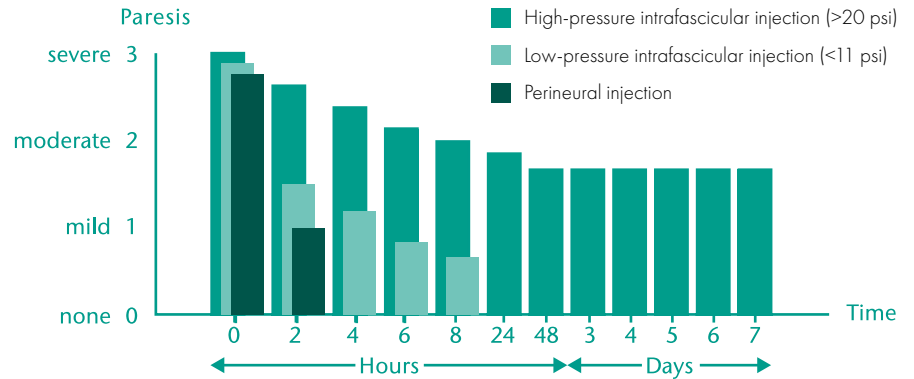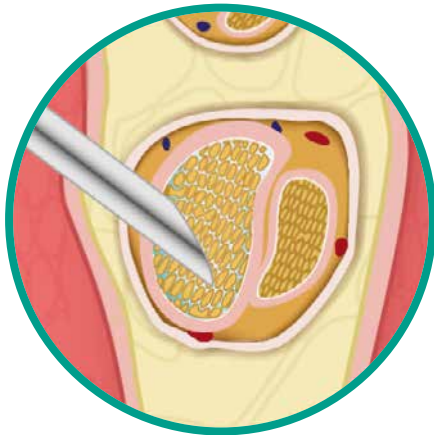



The higher the injection pressure is during injections in the intrafascicular space, the more severe and longer lasting the resulting paresis is.<sup>5</sup>

If the pressure during intrafascicular injections exceeds a critical threshold of 15 psi, such injections may lead to severe long-term neurological complications.<sup>5,6,7</sup> (Kapur <sup>6</sup>: > 20 psi, Hadzic <sup>6</sup>: > 25 psi, Hasanbegovic <sup>7</sup>: > 15.9 psi)


## Needle-Nerve Contact

**Effects:** Direct needle-nerve contact can lead to damage of the neural structures with subsequent transient or permanent neurological impairment.<sup>9,10</sup> Localization control using ultrasound and/or nerve stimulation may not in all cases reliably indicate direct needle-nerve contact. Paresthesia in and of itself is not regularly observed.<sup>9</sup> Avoiding direct needle-nerve contact minimizes the risk of damage to the nerve wall.<sup>10</sup>

→ A reliable indicator of direct needle-nerve contact is a high opening pressure

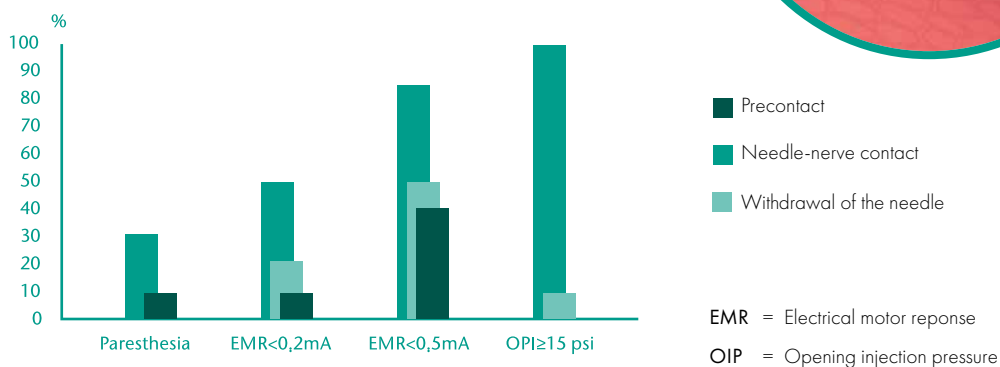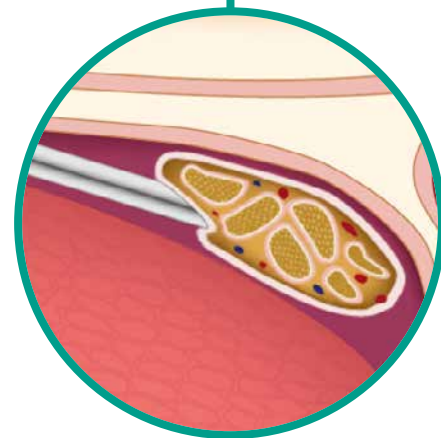

Incidence of paresthesias, motor evoked responses as well as exceeding the opening pressure threshold in the case of three different needle positions.<sup>9</sup>

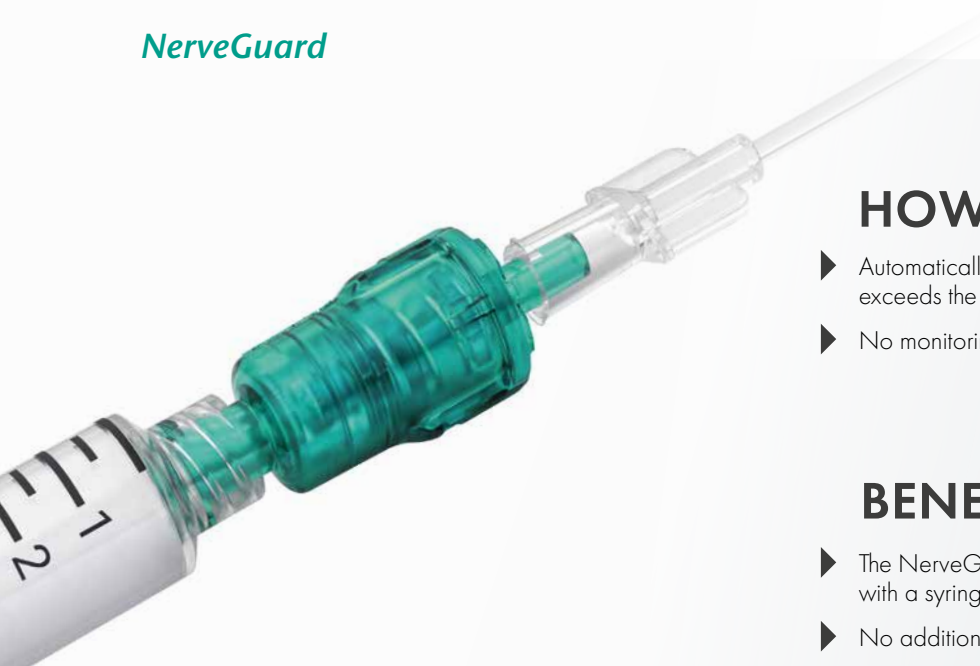

## HOW IT WORKS

- ▶ Automatically blocks injections when injection pressure exceeds the limit value<sup>12,13,14,15</sup>
- ▶ No monitoring and no visual control necessary



## BENEFITS

- ▶ The NerveGuard injection pressure limiter is easily connected with a syringe and injection tube
- ▶ No additional „eye contact“ with the NerveGuard is necessary

Description	Item no.	Item no. NRFit®	PU
NerveGuard / single / sterile / For single shot applications in diameters of 22G / 21G / 22G	001151-38M	001163-38M	10

## Studies

1. Choquet O., Capdevila X. Ultrasound-guided nerve blocks: the real position of the needle should be defined, *Anaesth. Analg.* 2012 May; 114(5): 929–930
  2. Neil J. M., Brull R., Horn J. L., Liu S. S., McCartney C. J., Perlas A., Salinas F. V., Tsui B. C. The Second American Society of Regional Anesthesia and Pain Medicine Evidence-Based Medicine Assessment of Ultrasound-Guided Regional Anesthesia, *Reg. Anaesth. Pain Med.* 2016 March–April; 41(2): 181–194
  3. Robards C., Hadzic A., Somasundaram L., Iwata T., Gadsden J., Xu D., Sala-Blanch X. Intraneural injection with low-current stimulation during popliteal sciatic nerve block, *Anesth.* 2009 Aug; 109(2): 673–677
  4. Sites B. D., Spence B. C., Gallagher J. D., Wiley C. W., Bertrand M. L., Blike G. T. Characterizing novice behavior associated with learning ultrasound-guided peripheral regional anesthesia, *Reg. Anesth. Pain Med.* 2007 Mar–Apr; 32(2): 107–115
  5. Kapur E., Vuckovic I., Dilberovic F., Zaciragic A., Cosovic E., Divanovic K. A., Mornjakovic Z., Babic M., Borgeat A., Thys D. M., Hadzic A. Neurologic and histologic outcome after intraneural injections of lidocaine in canine sciatic nerves, *Acta. Anaesthesiol. Scand.* 2007 Jan; 51(1): 101–107
  6. Hadzic A., Dilberovic F., Shah S., Kulenovic A., Kapur E., Zaciragic A., Cosovic E., Vuckovic I., Divanovic K. A., Mornjakovic Z., Thys D. M., Santos A. C. Combination of intraneural injection and high injection pressure leads to fascicular injury and neurologic deficits in dogs, *Reg. Anesth. Pain Med.* 2004 September–October; 29(5): 417–423
  7. Hasanbegovic I., Kulenovic A., Hasanovic S. Effects of intraneural and perineural injection and concentration of ropivacaine on nerve injury during peripheral nerve block in wistar rats, *J. of Health Sciences* 2013; 3(3): 243–249
  8. Gadsden J., Choi J. J., Lin E., Robinson A. Opening injection pressure consistently detects needle-nerve contact during ultrasound-guided interscalene brachial plexus block, *Anesthesiology* 2014 May; 120(5): 1246–1253
  9. Gadsden J., Latmore M., Levine D. M., Robinson A. High Opening Injection Pressure is Associated With Needle-Nerve and Needle-Fascia Contact During Femoral Nerve Block, *Reg. Anesth. Pain Med.* 2016 Jan–Feb; 41(1): 50–55
  10. Steinfeldt T., Graf J., Schneider J., Nimphius W., Weihe E., Borgeat A., Wulf H., Wiesmann T. Histological consequences of needle-nerve contact following nerve stimulation in a pig model, *Anesth. Research and Practice* 2011; Article ID 591851: 0–9
  11. Lundborg G., Myers R., Powell, H. Nerve compression injury and increased endoneurial fluid pressure: a „miniature compartment syndrome“, *J. Neurol. Neurosurg. Psychiatry* 1983 Dec; 46(12): 1119–1124
  12. Gadsden, J. (2018) Current devices used for the monitoring of injection pressure during peripheral nerve blocks, *Expert Review of Medical Devices*, 15:8, 571-578.
  13. Herreweghe, I.V. et al. (2020) Injection Pressure Monitoring Devices: A Review by NYSORA-Europe, *Anesthesiology News Special Edition* 2020, 101-104.
  14. McLeod, G.A. (2021) Novel approaches to needle tracking and visualisation, *Anaesthesia* 2021, 76 (Suppl. 1) 160-170.
  15. Rambhia, M., Gadsden J. (2019) Pressure Monitoring: The evidence so far, *Best Practice & Research Clinical Anaesthesiology*, <https://doi.org/10.1016/j.bpa.2019.03.001>
- Adams A. Injection Pressure Monitoring & Injection Technique in Peripheral Nerve Blockade, *Anaesth. J. of the AAGBI* 2015 Nov; 70(2)
  - Patil J., Ankireddy H., Wilkes A., Williams D., Lim M. An improvised pressure gauge for regional nerve blockade/anesthesia injections: an initial study, *J. Clin. Monit. Comput.* 2015 Dec; 29(6): 673–679
  - Patil J. J., Ford S., Egeler C., Williams D. J. The effect of needle dimensions and infusion rates on injection pressures in regional anaesthesia needles: a bench-top study, *Anaesth.* 2015; 70: 183–189
  - Ross S., Edwards K., McFadden K., Bigeleisen P. E., Orebaugh S. L. Pressures of Injection in a Cadaver Model of Peripheral Nerve Blockade, *J. Anesth. Clin. Res.* 2014; 5: 10

PAJUNK® GmbH Medizintechnologie  
Global Headquarters  
Karl-Hall-Straße 1  
78187 Geisingen, Germany  
Tel.: +49 7704 9291-0  
Fax: +49 7704 9291-600  
info@pajunk.com  
pajunk.com

PAJUNK® Medical Produkte GmbH  
Central Europe  
Karl-Hall-Straße 1  
78187 Geisingen, Germany  
Tel.: +49 7704 8008-0  
Fax: +49 7704 8008-150  
info@pajunk-medipro.com  
pajunk.de & pajunk.eu

PAJUNK® Swiss AG  
Switzerland  
Neuhofstrasse 5A  
6340 Baar, Switzerland  
Tel.: +41 417 609 627  
info@pajunk.ch  
pajunk.ch

PAJUNK® Medical System L.P.  
United States of America  
4575 Marconi Drive  
Alpharetta, GA 30005, USA  
Tel.: +1 888 972 5865  
Fax: +1 678 514 3388  
info@pajunk-usa.com  
pajunkusa.com

PAJUNK® UK Medical Products Ltd  
United Kingdom  
Unit D1 The Waterfront  
Goldcrest Way, Newburn Riverside  
Newcastle upon Tyne, NE15 8NY, UK  
Tel.: +44 191 264 7333  
info@pajunk.co.uk  
pajunk.co.uk